VistaScan Perio Net

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VistaScan Perio Net

Only fully-qualified personnel may set-up, install or operate this appliance.

Minimum requirements

Imaging program	DBSWIN or VistaEasy by Dürr Dental Third-party software on request
Computer system	See "System requirements for computer systems for imaging" (9000-618-148/01)

1 Connecting up the device

- Connect the appliance to the network using the network cable supplied.
- Switch the device on.


2 Installation

- Switch on the network devices (router, PC, and switch).
- Check that TCP port 2006 and UDP port 514 are released in the firewall being used; if not, release them.
If you are using the Windows firewall, you do not need to check the ports since you are asked if you want to release them during the driver installation process.


3 Configuration

Configuration is carried out using VistaNetConfig, which is automatically installed on installation of DBSWIN or VistaEasy.

- Click *Start > All Programs > Dürr Dental > VistaConfig > VistaNetConfig*.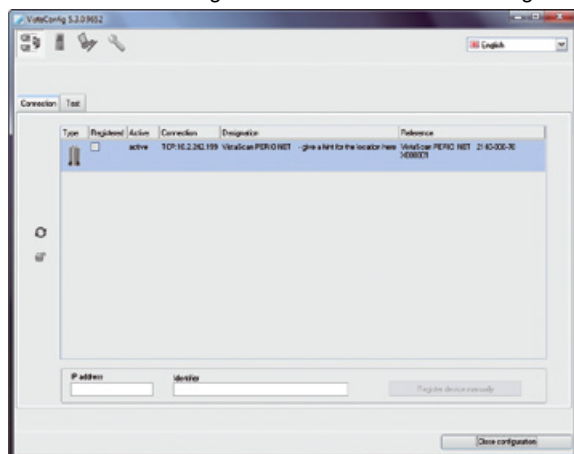

- Click the "Refresh" symbol
- Activate the connected device in the "Registered" column. You can also register multiple devices.
- Click the "Edit" button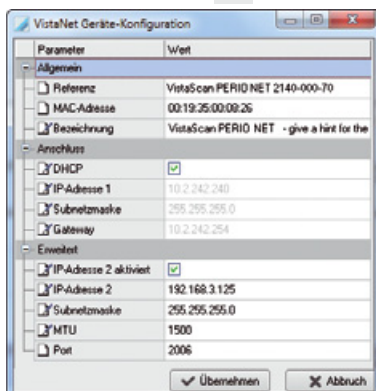

You can change the device name, manually enter an IP address, and query information in the "VistaNet device configuration" window.


Entering a permanent IP address (recommended)

- Deactivate DHCP.
- Enter an IP address, subnet mask, and gateway.
- Click "Apply" to save the configuration.


4 Testing the device

You can read in an X-ray image to test whether or not the device is connected up properly.

- Go to the "Test" tab.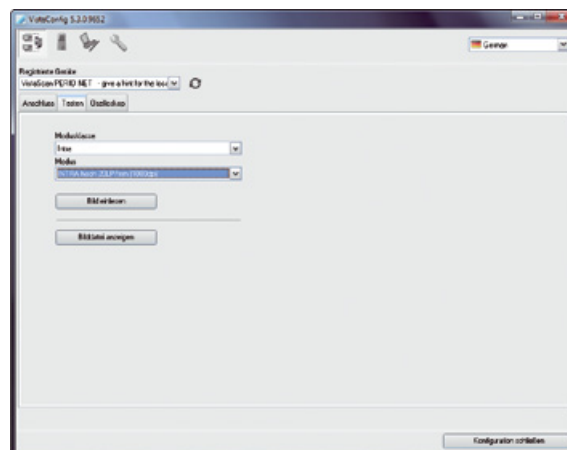

- Select the device from the "Registered Devices" selection list.
- Select the mode class.
- Select the mode.
- Click "Read image".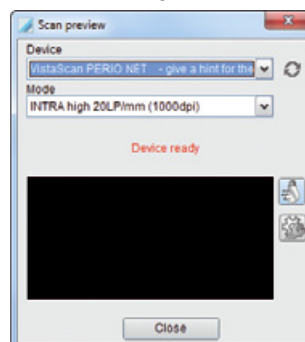

- Read in the image plate.


To scan image data, refer to the VistaScan Perio Net installation and operating instructions (order number 9000-618-70/..).

5 Check amount of light exposure

The oscilloscope can be used to measure the amount of light exposure to the unit.

- Choose the „Oscilloscope“ option.
- Select the device from the "Registered Devices" selection list.
- Slide the regulator bar for the HV value to 650.
- Click „Start/Stop“ to begin measurements.
- Click „Start/Stop“ to end measurements.
- Pull the left hand slider bar downwards until all three lines can clearly be seen.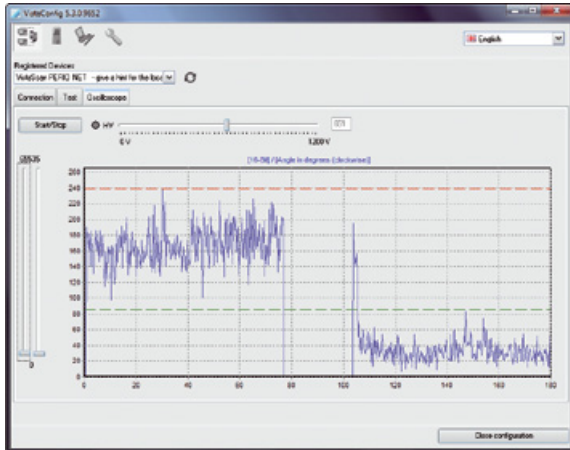

Blue line: Measured light exposure

Red line: Maximum value

Green line: Average value

The test has been passed successfully when the measured value does not exceed 450.

- When the maximum value is higher than 450, reduce the amount of room lighting.

6 Electrical safety check

- Carry out the electrical safety check in accordance with national and local regulations.
- Document the results.