VistaRay Configuration

D

1. Installation

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GB

1. Installation

Place the CD-ROM (e.g. DBSWIN from Version 3.4b) supplied into the drive. Interrupt the automatic Setup procedure.

2. USB connections

VistaRay is connected via a USB-interface (1.1 oder 2.0). The following operating systems support this port.
Windows 2000: USB 1.0 or USB from Service Pack 4
Windows XP: USB 1.1 or USB2 from Service Pack 2

Check the system properties

- Open system properties (right hand mouse click on My Computer, Properties).
- Check the version number of the Service Pack and update if necessary.

Check USB interface

- Open hardware manager (right hand mouse click on My Computer, Properties, Hardware, device manager).
- „USB-Controller“ must be in the hardware list.

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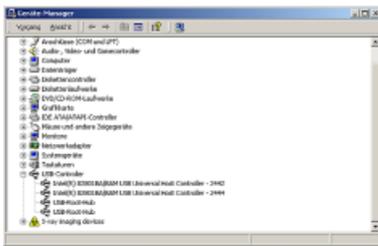
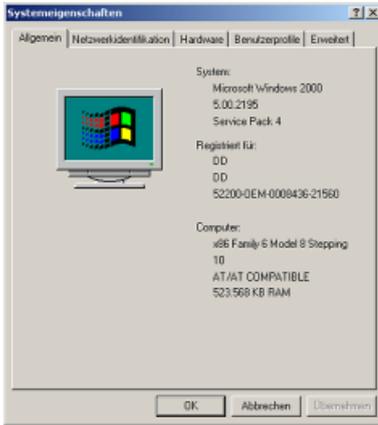
Connecting VistaRay

See Installation and Operating Instructions for VistaRay 5 (Version USB) 9000-618-11/xx

Install USB driver

- Windows recognises a new appliance. (USB-Converter 2121-140-51).

- Select the option for manual installation and confirm with „Continue“. Windows 2000, XP: „Find suitable driver“

- Select the option for manual installation of driver and confirm with „Continue“. Windows 2000, XP: „CD-drives“.

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- Confirm choice of driver with „Continue“

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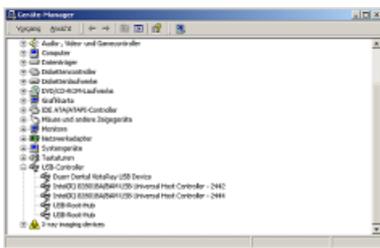
Check the driver in device manager

- Open hardware manager (right hand mouse click on My Computer, Properties, Hardware, device manager).
- Check is positive when the driver Dürr Dental VistaRay USB has been listed in the appliance list.

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3. Configuration

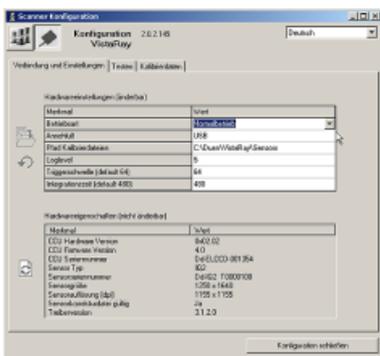
Program start

- Click the following in sequence [Start], [Program], [VistaScan] and [VistaScanConfig].

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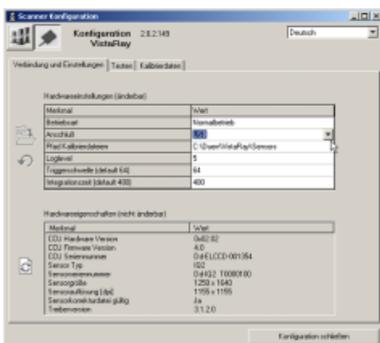
Dialogue field [Connections]

- In the menu [Operating Mode] select the setting „Normal operation“.

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
- In the menu [Connections] select the interface being used.

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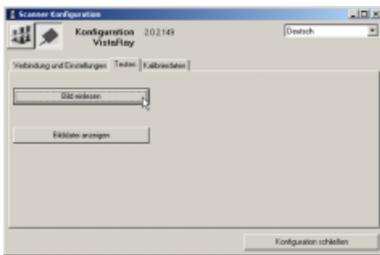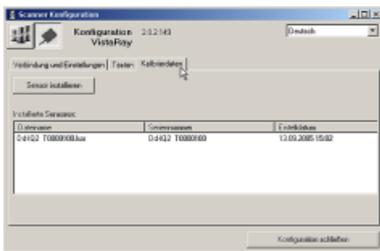
Dialogue field [Test]

- Click on option [Scan image].

- Set resolution.
- Expose the CCD-Sensor to X-ray beams as explained in the Installation and Operating Instructions (9000-618-11/xx).
- The test has been successful when the image appears.
- Close the window.
- Save the X-ray exposure for documentation purposes.

Dialogue field [Calibration]

- A list of the CCD-sensors and the appropriate calibration data is to be found here.
- If there is no calibration data file for a particular sensor in the list, then this can be subsequently installed using the sequence and the dialogue „Install sensor“.

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4. Installation

To install the DBSWIN image processing software place the installation CD-ROM in the drive. If the program does not start automatically, open the file setup.exe from the CD-ROM. Install the program DBSWIN according to the instructions on the screen.

i Where VistaRay is being operated using DBSWIN or via VistaRay Easy, then X-ray exposures can be carried out either using a high resolution mode (22.7 LP/mm) or under standard mode (11.35 LP/mm).

Transmission time is between 4 and 16 seconds depending on the size of the sensor and the settings.

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Program start

- Click the following in sequence [Start], [Program], [VistaScan] and [VistaScanConfig].
- Start DBSWIN without a password as user [Standard].

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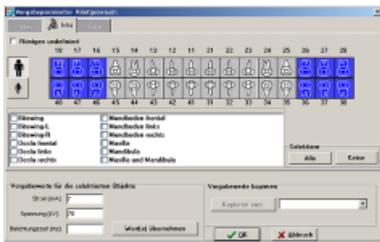
5. Configuration

Module [VistaRay]

- In menu [Options] select the line [Display configuration].
- Click on [modules] on the left hand option symbol bar.
- Use the dialogue field [Standard mode] to set the desired resolution for X-ray images.

Use the dialogue field [Configuration program] to start VistaRayConfig.

- Check under [Appliance-IDs] the initialisation of VistaScan and successful insertion of the serial number.

Module [X-ray]

- Click under [X-ray Station] on the symbol [New X-ray Station] on the right hand symbol panel.
- Enter the parameters required in the fields provided.

- Click on option [Set parameters].
- Enter for all the possible exposure situations the best exposure times for the X-ray device.

See Table „Recommended exposure times“ in the Installation and Operating Instructions 9000-618-11/xx

- Confirm each parameter setting with [Accept Values] and to finish with [OK].

Refer also to the DBSWIN Handbook in the menu [Help], [Contents], [Configuration]

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