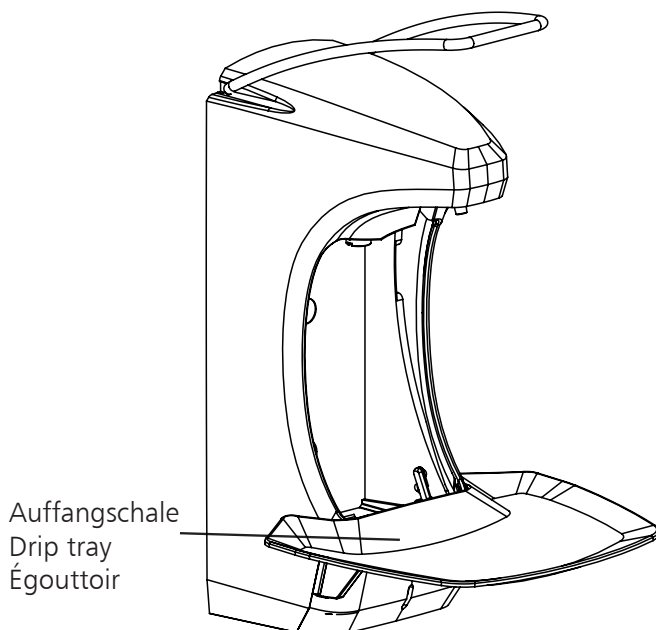
# RX 5 M

Seifen- und Desinfektionsmittelspender  
Dispenser for liquid soaps and disinfectants  
Distributeurs de savon et de désinfectant

## **Wichtig! Important!**

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The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. OPHARDT hygiene assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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## EN Instruction manual

1. Indications for the use of the precautions instructions	12
2. Intended use.....	12
3. Safety guidelines.....	12
4. Technical drawing.....	13
5. Delivery check list.....	13
6. Mounting the dispenser.....	14
7. Mounting the drip tray.....	15
8. Range of OPHARDT Hygiene pumps for RX 5 M.....	15
9. Going into operation.....	16
10. Moving the adjustable bottle ledge.....	17
11. Basic operating.....	18
12. Maintenance and care.....	19
13. Warranty.....	19
14. Disposal.....	19
15. Spare parts list.....	19

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## 1. Indications for the use of the precautions instructions / 2. Intended use / 3. Safety guidelines

### 1. Indications for the use of the precautions instructions

The following symbols are used in the instructions

#### **DANGER**

This symbol warns the reader that subsequent information is vital to avoiding significant injury. Failure to observe this piece of information is likely to result in a serious injury that could be fatal.

#### **WARNING**

This symbol draws the reader's attention to information that is useful to avoiding accidents that can result in a significant injury. Failure to observe this piece of information can result in a serious injury that could be fatal.

#### **CAUTION**

This symbol draws the reader's attention to a potentially hazardous situation. Failure to observe this information can result in injury or damaged property.

#### **NOTICE**

This symbol draws the reader's attention to important information that will facilitate the proper handling of the described product. Failure to observe this information may result in malfunction of the product or property damage.

### 2. Intended use

- This dispenser is designed for the dispensing of hand hygiene liquids.
- Suitable for liquid soap, hand disinfectant, alco-gel and flowable lotions.
- It is **not** intended for use with high-viscosity products or abrasive and aggressive agents.
- For use with unapproved liquids, please contact OPHARDT Hygiene. Details on the back of this booklet.
- A RX 5 M locking version (Art.-No.4401842) and non-locking version (Art.-No. 4401841) is available.

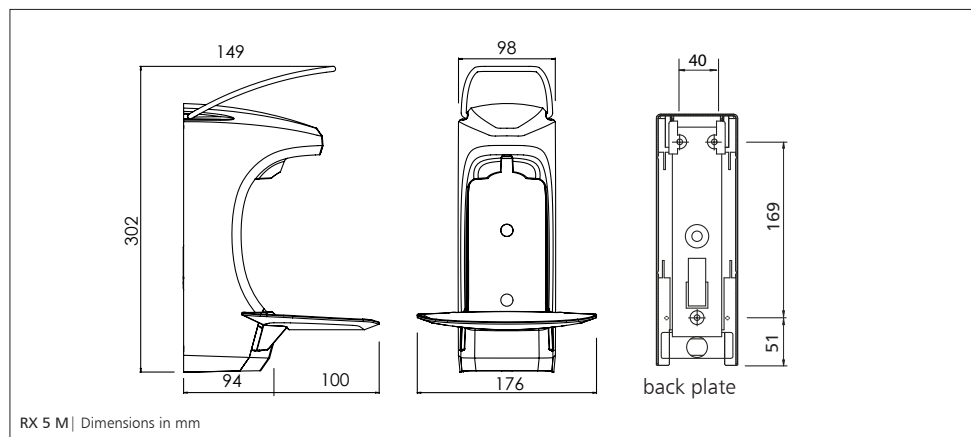
### 3. Safety guidelines

#### **CAUTION**

- For soap applications the dispenser can be mounted over a sink or wash area. For disinfectant, the dispenser can be mounted on any flat wall.
- Do not place the device directly in high-traffic routes.
- Ensure that the drip tray is empty when opening the dispenser

## EN 4. Technical drawing / 5. Delivery check list

### 4. Technical drawing


### 5. Delivery check list

- Dispenser
- Pump
- Bottle
- Drip tray
- Instruction manual
- ingo-man® standard back plate and installation set
- Key (only with lockable version Art.-No. 4401842)

**EN 6. Mounting of the dispenser****6. Mounting the dispenser**

This dispenser is for wall-mounting only.

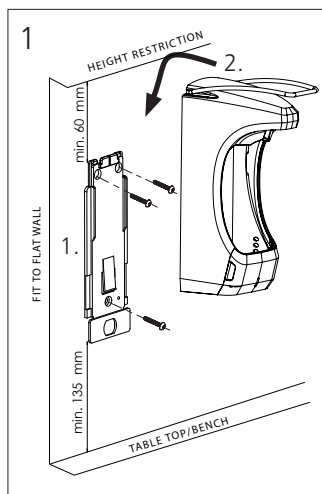
Follow the instructions listed below:

**CAUTION**

- Do not install the appliance in narrow entrances and passageways! Mount the dispenser 1.2m from the floor to the bottom of the dispenser

**NOTICE**

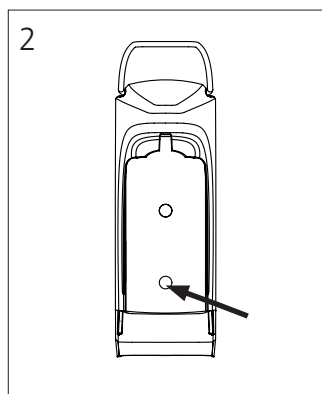
- Check the suitability of the wall before mounting!
- When choosing where to mount the appliance, please consider ease of access for filling! Allow 60 mm min. distance at the top of the back plate. Allow 135 mm min. distance from the bottom of the back plate to a table top or bench.
- The back plate must be mounted to a flat wall using the 3 screws provided. (Fig. 1)
- The dispenser should be fitted against the back plate and moved vertically down to engage the backplate lock. (Fig. 1)
- To remove the dispenser from the backplate, press plastic spring tab through hole on the dispenser and lift upwards. (Fig. 2)

## 7. Mounting the drip tray / 8. Range of OPHARDT Hygiene pumps for RX 5 M
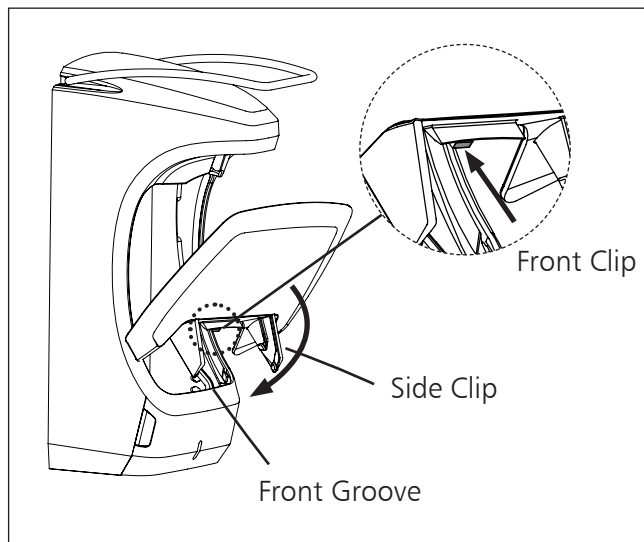
### 7. Mounting the drip tray

1. Allow the drip tray front clip to engage on the front groove.
2. Swing the drip tray in a downward arc and allow the side clip to lock into position on the front groove.
3. The drip tray is designed to detach from the dispenser in the event of an impact.

### 8. Range of OPHARDT Hygiene pumps for RX 5 M

The RX 5 M dispenser can dispense a wide variety of liquids. OPHARDT Hygiene pumps are developed to handle all types of liquids, soaps, gels and alcohols. The pumps which are suited for this dispenser include:

- RX 5 K pump, standard plastic pump (Art.-No. 4401434)
- DHP RX, disposable hygiene pump (Art.-No. 4401860)
- Compatible with the OPHARDT Hygiene package

**EN 9. Going into operation**

## 9. Going into operation

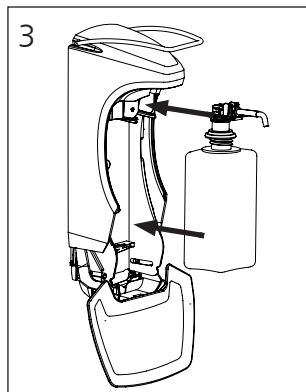
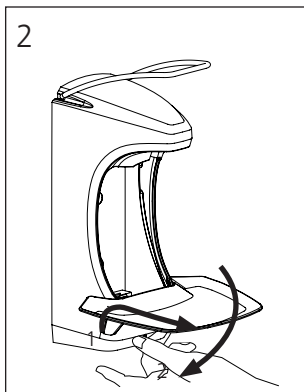
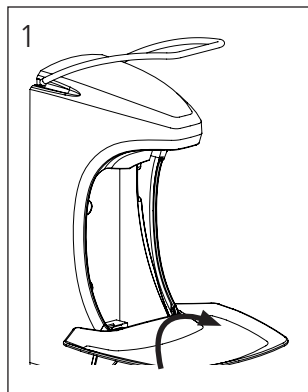
### Insert the bottle

**NOTICE**


- Only open this dispenser when it is fitted to the back plate on the wall

**CAUTION**

- Ensure that the drip tray is empty when opening the dispenser

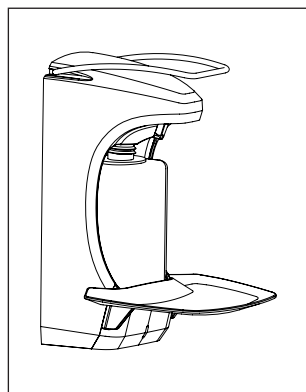1. (Locking Dispenser: Art.- No. 4401842) Place key provided in lock slot and turn 90 deg. (Fig. 1) (FOR KEYED VERSION ONLY)
2. Place fingers under lifter. Lift and gently pull forward to open. (Fig. 2)
3. Rotate the drip tray and lifter downward to access the pump and bottle compartment.
4. Slide pump and 0.5 L bottle into position as shown. (Fig. 3)
5. Close the dispenser and turn key 90deg.


## EN 10. Moving the adjustable bottle ledge

### 10. Moving the adjustable bottle ledge.

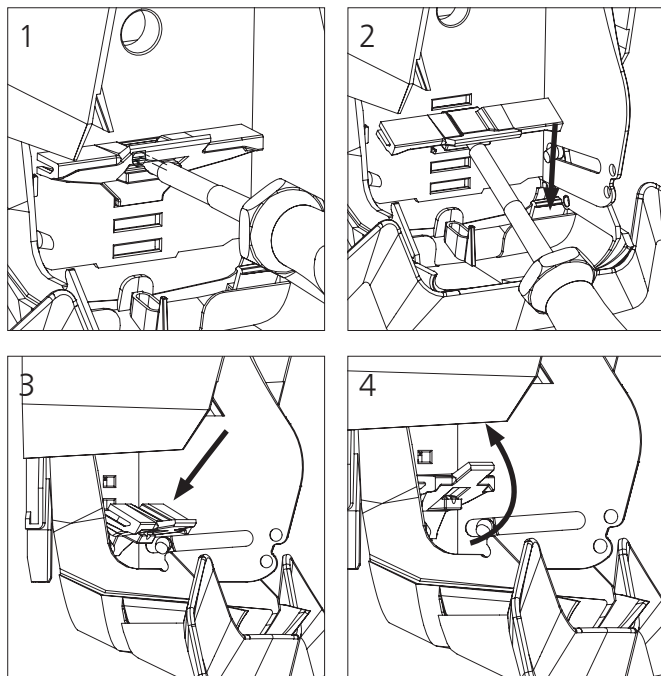Adjustable bottle ledge

**Note: This dispenser has three positions for the adjustable bottle ledge.**


This is to suit a wide variety of bottle heights.

**Refit the adjustable bottle ledge into a suitable position.**

1. Open the dispenser.
2. Insert screwdriver as shown into the adjustable base angle. (Fig. 1)
3. Pivot screwdriver downwards and the base angle will disengage from the dispenser. (Fig. 2)
4. Move into new desired position inserting the bottom clip first as shown. (Fig. 3)
5. Swing adjustable base angle up and lock it into position. (Fig. 3 and 4)

EN 11. Basic operating
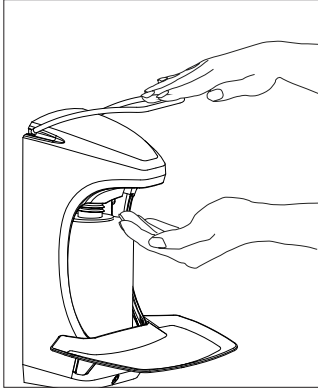
## 11. Basic operating

The user should place their open hand under the outlet nozzle.

Press down on the lever to operate the dispenser.

For initial priming, the unit must be operated several times until a full dosage is dispensed.

EN 12. Maintenance and care / 13. Warranty / 14. Disposal / 15. Spare parts list

## 12. Maintenance and care

Please follow the recommendations of the Robert Koch-Institute for cleaning ([www.rki.de](http://www.rki.de)).

### Cleaning the Dispenser

Wipe with a clean cloth. Only use water when cleaning this product

Disinfection of the shroud is possible with appropriate surface disinfectants.

Do not use harsh chemicals, caustic purifiers or scouring powders.

The RX 5 M can be removed from the wall and placed in a dish-washer for cleaning.

Max temp. 60 °C

**NOTICE**

- The appliance is not autoclavable!

## 13. Warranty

This appliance comes with a 2-year manufacturers-warranty from the date of delivery.

## 14. Disposal

- Dispose of waste packaging in accordance with legislation.
- All parts of this dispenser can be recycled.
- Please dispose of properly so that recycling is possible.

Materials:

- ABS plastic
- POM plastic
- Stainless steel

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OPHARDT HYGIENE-TECHNIK GmbH & Co. KG