
# RX 5 T

Berührungsloser Seifen- und Desinfektionsmittelspender  
Touchless dispenser for liquid soaps and disinfectants  
Distributeurs de savon et de désinfectant sans contact

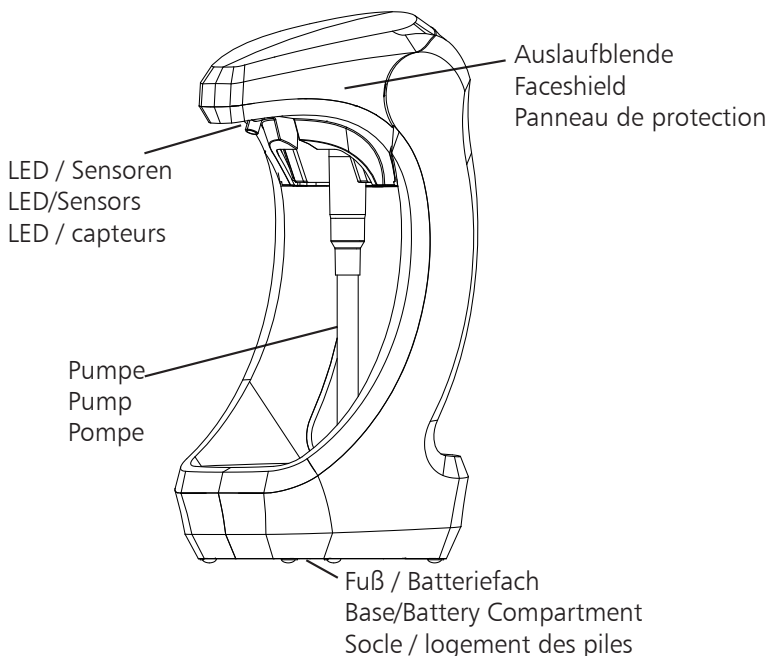## **Wichtig! Important!**

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The compatibility of filling material and dispenser system must be checked prior to the use of new filling materials, in order to be able to guarantee fault-free functionality. OPHARDT hygiene assumes no liability for any damages or costs which arise due to the incompatibility of filling material and dispenser systems.

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**EN** Instruction manual

1. Indications for the use of the service instructions ..	17
2. Intended use.....	17
3. Safety information.....	18
4. Technical drawing.....	19
5. Scope of delivery.....	19
6. Mounting of the dispenser.....	20
7. Going into operation.....	22
8. Basic operation.....	24
9. Changing stroke.....	25
10. LED Function.....	25
11. Maintenance and attendance.....	26
12. Troubleshooting.....	27
13. Warranty.....	28
14. Disposal.....	28
15. Technical data.....	29

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**EN 1. Indications for the use of the service instructions / 2. Intended use**

## 1. Indications for the use of the service instructions

The following symbols are used in the instructions

**Safety reference:**

A non-observance may cause personal injury

**NOTICE**

Indicates a property damage message

## 2. Intended use

This appliance is suitable for hand disinfectant and soaps.

It is **not** intended for use with high-viscosity products or abrasive and aggressive agents.

The dispenser is for free standing, wallmount or pillar mount use. Use this appliance indoors only.

Use the appliance for hand disinfection and cleaning, under normal use cycles.

**Ambient conditions:**

-  • Never use the appliance in potentially explosive areas.
-  • Never use the appliance in a dusty environment.
-  • Do not use the apparatus if the relative air moisture is higher than 90% (not condensing).
-  • This apparatus should not be used at room temperature below 15 °C or higher than 30 °C

## EN 3. Safety information

### 3. Safety information

#### Attention!

Please read carefully these operating instructions prior to initial operation and retain for future reference.

**OPHARDT hygiene** does not assume any liability for damages in the case of:

- defects at the appliance caused by mechanical influences
- changing of the appliance without express approval of **OPHARDT hygiene**
- use for other purposes than that mentioned in the application

#### 3.1 General safety


- ⚠ • Repairs of the appliance may exclusively be carried out by **OPHARDT hygiene** employees or persons authorized by **OPHARDT hygiene**. Appliance may become hazardous if tampered with. In case of a necessary repair please contact the manufacturer.
- ⚠ • Place the appliance in a dry and splash-proof place.
- ⚠ • This appliance is not a toy. Keep and use it out of the reach of children. The appliance should be installed where children cannot reach it.
- ⚠ • Children or individuals with physical, sensory or mental impairment should be supervised when using this appliance.
- ⚠ • The appliance is not suitable for the transportation of media.
- ⚠ • Do not place the device directly in traffic routes.
- ⚠ • The appliance is not suitable for the metering of other than the indicated media.
- ⚠ • Exclusively operate the appliance with leakproof alkaline batteries of the same manufacturer and type and with the same charging condition. Only use batteries with the indicated size type C.
- ⚠ • Do not operate the appliance with rechargeable batteries (accumulators).
- ⚠ • Do not use the dispenser if batteries have leaked in the appliance.
- ⚠ • Never use force.
- ⚠ • Take the batteries from the appliance prior to cleaning and maintenance.
- ⚠ • The appliance need not be disassembled, as there are no repairable parts.
- ⚠ • The face shield is not removable.
- ⚠ • The battery supply clamps may not be short-circuited with each other.
- ⚠ • Cleaning and user maintenance tasks must not be carried out by unsupervised children.

## EN 3. Safety information / 4. Technical drawing / 5. Scope of delivery

### 3.2 Safety during operation of the appliance

- ⚠ • Protect your appliance against humidity.
- ⚠ • Do not expose the appliance to thermal radiation and heat.
- ⚠ • Never submerge the appliance in water.
- ⚠ • Only fill the appliance with the indicated substances in accordance with the intended use.
- ⚠ • There are no removable parts with the exception of pump, receptacle, battery cover and batteries.
- ⚠ • The appliance is not autoclavable and cannot be cleaned in dishwasher.
- ⚠ • The apparatus should not be cleaned with water jets or by means of a high-pressure cleaner.

### 4. Technical drawing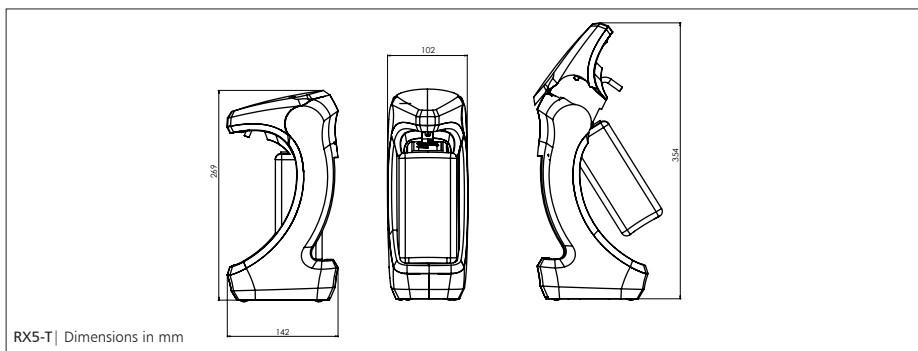

### 5. Scope of delivery

- Dispenser
- Plastic pump
- Receptacle, 500 ml
- Batteries (4 mono-cells, "type C")
- Instruction manual
- Optional: Wallmount (including ingo-man® standard backplate and fixing set)

## EN 6. Mounting of the dispenser


# 6. Mounting the dispenser

## 6.1 Wall mounting

The dispenser has the option of being free standing or wall mounted.

If free standing please ensure that:


- The surface is level and secure
- It is clear of any passing or moving objects
- If the dispenser is knocked any spillage should not affect nearby equipment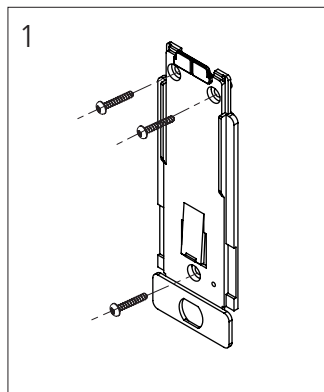

If the dispenser is to be mounted on a wall please follow the instructions listed below:

### NOTICE


- Check the suitability of the wall before mounting!
- Do not install the appliance in narrow entrances and passageways!
- When choosing where to mount the appliance, please consider ease of access for filling!
- Mount the appliance on the wall using the enclosed fixings or other suitable fixings.
- The dispenser must be mounted to the fixing apertures provided on a non-sloping wall using at least 3 screws.
- Microwaves or fluorescent tubes can cause electromagnetic interference. If this is the case, please remove the source of the interference. This will restore the device to problem-free operation.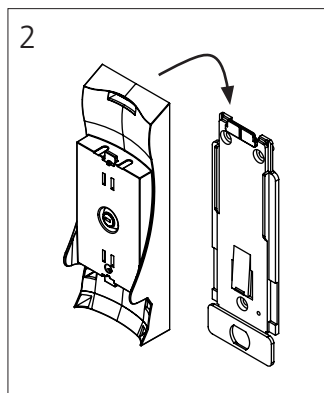


Fix the ingo-man backplate to the wall using the wall plugs & screws provided (fig. 1).

Push the wall mount to the wall and slide down over ingo-man backplate until the spring tongue is engaged (fig. 2).


## EN 6. Mounting of the dispenser

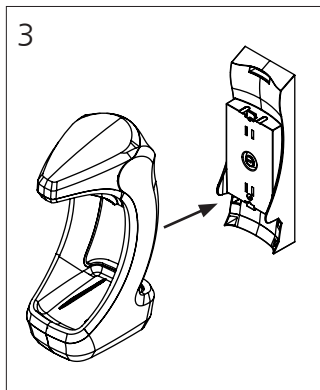Place the dispenser on the wallmount, lining up the bottle opening with the rectangular boss on the wallmount (fig. 3).

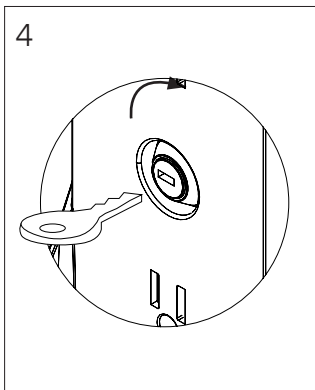
Turn the key to lock the dispenser in place. Remove the key before inserting the bottle (fig. 4).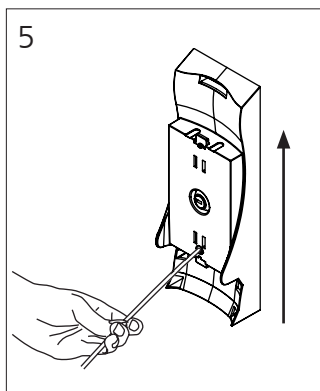

To remove the wallmount from the ingo-man backplate, using a thin tool push on the tongue of the backplate (through the hole provided) and slide the wallmount upwards (fig. 5).

## EN 7. Going into operation
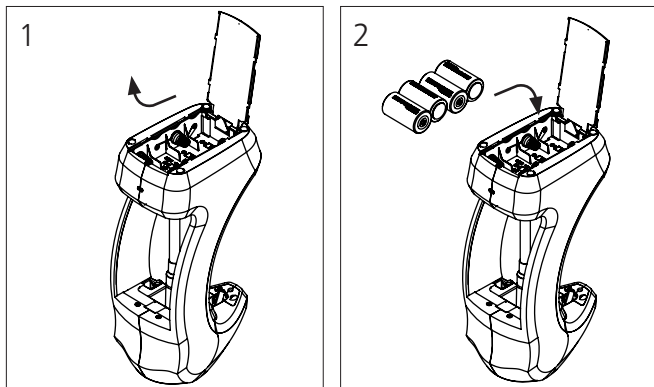
## 7. Going into operation

### 7.1 Insertion/Replacement of the batteries


Remove bottle and dispenser upside down to expose the battery compartment (fig. 1).

Open battery cover by sliding it to the back of the dispenser and then rotating it upwards.

Insert batteries as per marked on the battery compartment (fig 2).

Pay attention to the polarity.

**NOTICE** Batteries should be removed in the dispenser if not used for a prolonged period of time.
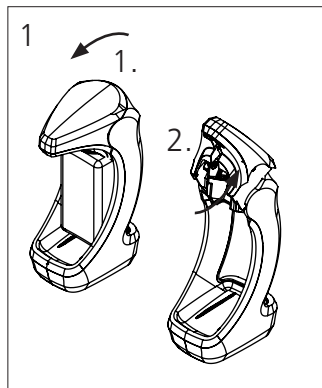


### 7.2 Inserting the pump & bottle



Open the faceshield by pulling forward (1.) and rotating it upwards and towards the rear of the dispenser (2.) (fig. 1).

**NOTICE** Remove the stored media before transport!

## EN 7. Going into operation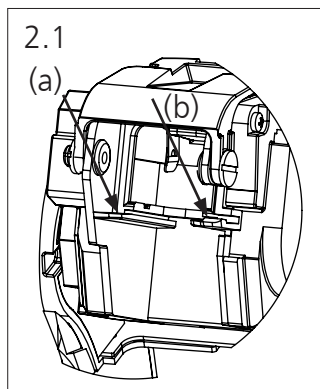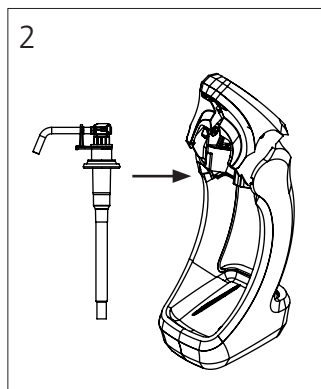

Slide the pump into the chassis guides (a) and past the lock tab (b) as shown (fig. 2 and fig. 2.1)
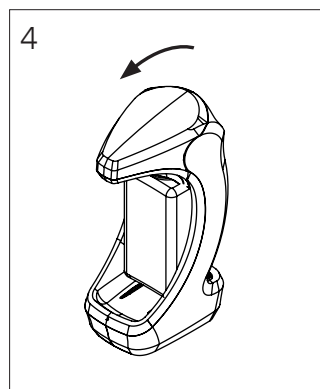
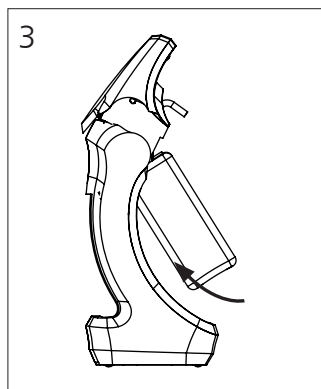


With the bottle at an angle slide the bottle up onto the pump. Push down on the front of the faceshield ensuring that it has closed correctly (see that back and sides match). Raising the bottle off the table or moving it to the edge of table may facilitate easier bottle installation.

EN 8. Basic operation

## 8. Basic operation

Once the faceshield is closed wait 3 seconds for red LED to go off.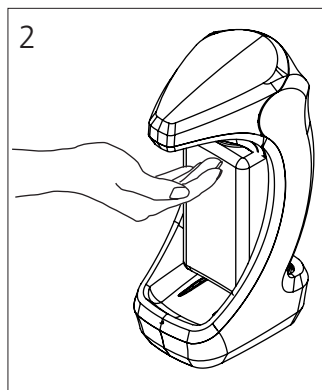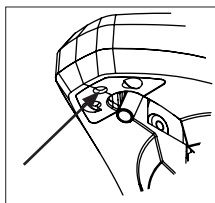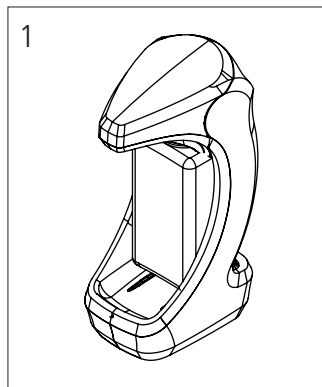

To activate the dispenser place the hand underneath the faceshield (approx. 5cm away).

This will trigger an automatic dispense. If the unit is set for multiple strokes (2 or 3) it will continue as long as the hand is present.

Dosage: 1,0 ml/stroke or 1,5ml/stroke (depending on the model)

EN 9. Changing dosage / 10. LED Function


9. Changing dosage

The dosage can be varied by selecting one, two or three strokes per activation. If set for multiple strokes the dispenser will only operate as long as a hand is present. If the hand is removed it will complete the current activation and then stop the dispenser.

The dispenser is supplied, set to single stroke as standard.

With the faceshield open press the tab (see 7.2) (left hand side) to increase the number of strokes.


The LED will flash amber when the tab is pressed correctly.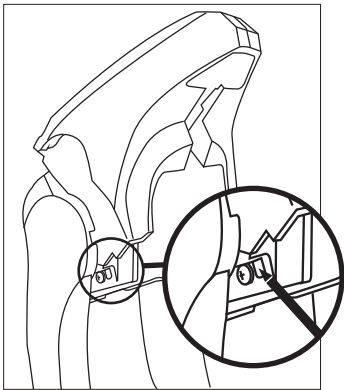

Pressing the tab	Number of flashes	Number of strokes
1st time	2 flashes	2 stroke
2nd time	3 flashes	3 stroke
3rd time	1 flash	Returns to 1 stroke

10. LED Function

Underneath the faceshield there is an LED which lights green, red and amber.

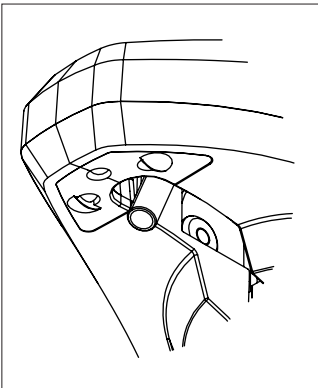Colour	Explanation
RED (Continuous)	Faceshield open/not closed correctly
RED (Flashing)	Batteries low
Green	Unit dispensing
Amber	Stroke selection (based on number of flashes)

**EN 11. Maintenance and attendance**

## 11. Maintenance and Attendance

Please consider that a high-quality product like this appliance requires attendance and care. The dispenser is maintenance-free with the exception of the battery exchange!

Please follow the recommendations of the Robert Koch-Institute ([www.rki.de](http://www.rki.de))

### **Cleaning / Disinfection dispenser:**

Please only use hot water and do not use caustic purifiers or scouring powders!

Remove the wallmount (if present) and the batteries as described in section 7.

The dispenser should only be cleaned from the outside. Wipe the dispenser with a damp cloth.

To ensure continuous functionality, an empty receptacle should be replaced as soon as possible.

 • **By no means disassemble the appliance!**

 • **Do not submerge the appliance or expose it to water jets!**

 • **The appliance is not autoclavable and cannot be cleaned in the dishwasher.**

### **Cleaning / Disinfection pump:**

 • **The pump itself is dishwasher-proof (60°C), however it must rest for about 6 hours after such a treatment before it is inserted again into the appliance.**

 • **If the suction valve becomes clogged, pump hot water through it in order to restore the function.**

 • **For disinfection please pump an alcohol based disinfectant through the system.**

## EN 12. Troubleshooting

## 12. Troubleshooting

 If the appliance malfunctions, please remove batteries or unplug it from the mains.

Characteristic	Possible cause	Solution
1. The appliance seems to function normally but no medium is dispensed.	<ul style="list-style-type: none"> <li>No receptacle is inserted.</li> <li>No pump is installed.</li> <li>Appliance is not closed</li> <li>Pump is clogged.</li> <li>Receptacle is empty.</li> <li>Pump is not primed for operation or has lost contact to the medium.</li> <li>An unsuitable medium has been used.</li> </ul>	<ul style="list-style-type: none"> <li>Insert full receptacle.</li> <li>Insert pump into the dispenser.</li> <li>Close the appliance</li> <li>Remove pump and clean it.</li> <li>Replace receptacle.</li> <li>Carry out the dosage process several times. If it doesn't function, the pump is possibly clogged.</li> <li>Use another medium.</li> <li>Clean the pump thoroughly before.</li> </ul>
2. The appliance doses too much or not enough medium	<ul style="list-style-type: none"> <li>The number of strokes has not been set correctly</li> </ul>	<ul style="list-style-type: none"> <li>Adjust the number of strokes. See section 9.</li> </ul>
3. The appliance doesn't dose	<ul style="list-style-type: none"> <li>Batteries are empty</li> <li>Batteries have not been inserted correctly.</li> </ul>	<ul style="list-style-type: none"> <li>Insert new batteries.</li> <li>Pay attention to the polarity</li> </ul>

If the issue cannot be resolved, please contact the manufacturer.

OPHARDT Hygiene-Technik GmbH & Co. KG  
Lindenau 27 | 47661 Issum | Germany  
Tel.: +49 2835 18-10 | Fax: +49 2835 18-77  
www.ophardt.com | E-Mail: customer care@ophardt.com

## 13. Warranty

This appliance comes with 2 years manufacturer-warranty from the date of delivery.

## 14. Disposal

Dispose of the packing material into the corresponding recycling plants.

The symbol on the product or its packaging indicates that this product should not be treated as standard household waste, but should be taken to a recycling point for electrical and electronic devices.

Further information may be obtained from your municipality, the municipal disposal companies or the company where you purchased the product.

### 14.1 Battery disposal

Batteries should not be disposed of as household waste. As a consumer, you are statutorily obliged to return used batteries. You can dispose of old batteries at domestic collection points, public collection points in your municipality or at any location where batteries of this type are sold.

## EN 15. Technical data

### 15. Technical data

Housing.....ABS

Receptacle.....synthetic material,  
removable 500 ml

Pump.....PP, PE & Silicone

Dosage.....1,0 ml/stroke or  
1,5ml/stroke  
(Model dependent.  
Please see label on the  
inside of the battery cover)

Voltage.....6 volt DC

Batteries.....4 x "type C"

Dimensions in mm.....W 102 x H 269 x D 142

Total weight.....0.930 kg  
(with batteries, without backplate)

Weight.....1.067 kg  
optional with backplate

Conformity.....RoHS compliant


Machinery directive 2006 / 42 / EG

EMC- directive 2004 / 108 / EG

EN 55014-1:2006

EN 55014-2:2001

EN 60335-1/A11:2014

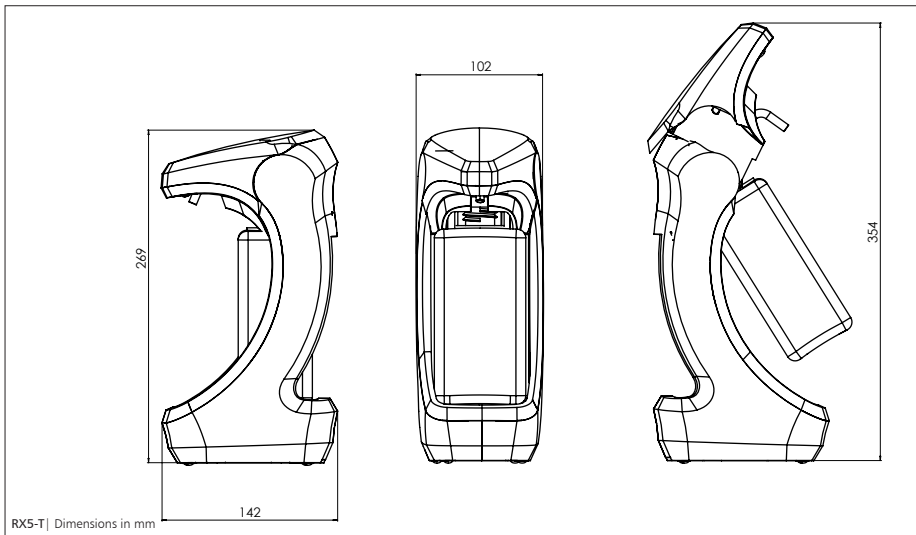Technical modifications reserved.  
Prior to the use of new filling media tests are required!

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OPHARDT HYGIENE-TECHNIK GmbH & Co. KG