Dürr VistaScan Perio

- D Folienkassette
- GB Foil cassette
- F Support de Plaque
- E Casete de laminillas o placas
- I Caricatori
- NL Foliecassettes
- P Cassete de película
- S Kassetter
- DK Foliekassette
- FIN Levykasetit
- NOR Foliekassett
- RUS Кассеты для рентгеновской плёнки
- GR Κασέτες φύλλων
- PL Kasety na płyty obrazowe

Bestellinformationen / Ordering Information

Size 0	2x3	1x	2140-010-00
Size 1	2x4	1x	2140-011-00
Size 2	3x4	1x	2140-012-00
Size 3	2,7x5,4	1x	2140-013-00
Size 4	5,7x7,6	1x	2140-014-00

2015/11SE

DÜRR DENTAL SE

Höpfigheimer Str. 17
74321 Bietigheim-Bissingen, Germany

Casete de laminillas o placas

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Foil cassette

The foil cassette is designed to ease the introduction of the image plate into the VistaScan. It also serves to hinder the accidental deletion of X-ray information on the image plate through stray ambient light.

Do not autoclave the foil cassette

Refer also to the Installation and Operating Instructions for the Dürr VistaScan Perio.

- Now tear the light protection cover open along the adhesive strip (V-indent).
- Choose a foil cassette of the correct size.
- Open the cover (9a) of the foil cassette until it clicks once.
- The opened light protection cover is now placed in slot 1 or slot 2. The inactive side (black, printed) must be visible.
- Slide the image plate out of the light protection cover and immediately into the foil cassette until it is firmly in position.
- Close the cover (9a) to the cassette.

Damage to the image plate:
Sliding the foil cassette incorrectly can lead to wear to the surface of the image plate. Therefore, the light protection cover must lie firmly and snugly in the cover of the foil cassette.

Loss of image data:
Image data held on the image plate will be lost on exposure to light. Exposed image plates should never be handled without either a light protection cover or a foil cassette.

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