

VistaCam iX HD Proof

Short instruction



2109100005



www.duerrendental.com









Caries view:

- Carry out teeth cleaning.



Prophylaxis view:

- **Don't** carry out teeth cleaning.

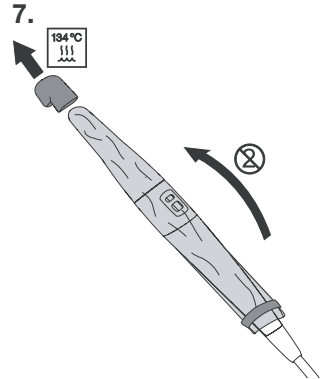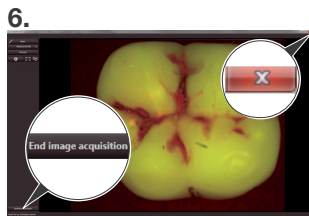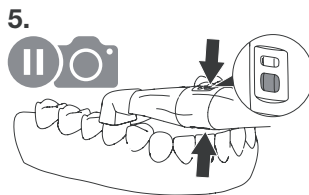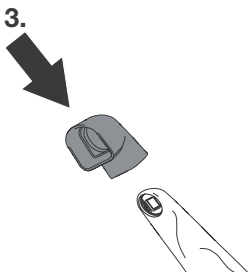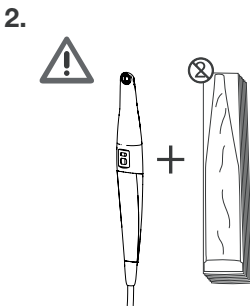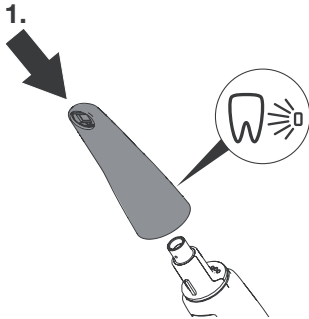

Analysis



The colour scale and the numeric values provide reliable information on carious lesions:



Healthy tooth enamel

1,0



Early-stage caries, incipient enamel caries

1,5



Enamel caries up to enamel/dentine junction

2,0



Dentine junction already exceeded

2,5



Deep dentine caries

DÜRR DENTAL SE

Höpfheimer Str. 17
74321 Bietigheim-Bissingen

2109100009L01

