Unit Monitor

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Power Tower PC monitoring software

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0949-100-51

 DÜRR
DENTAL

EN Installation and Operating Instructions

The software Unit Monitor enables user friendly monitoring of the Power Tower from any computer.

1. Installation

System requirement:

- Operating systems Microsoft Windows 7, XP or Vista
- Sound card with speaker
- Network with DHCP
- Computer and Power Tower connected to network

 Only sufficiently qualified staff should install the software and start operation.

- Place the CD-ROM in the CD-drive.
- Double click the *unit-monitor-install.exe* file on the CD-ROM.
- Follow instructions given by the installation assistant.

 Once installation is complete a link to the program will appear on the desktop monitor.

 In order that Unit Monitor will start automatically when the computer is started, the program is added to the autostart menu.

2. Operation

 The program must be activated and the loudspeakers switched on while the Power Tower is being operated.

2.1 Start program manually

- Double click the desktop link.

The status display will open. This is identical to the display on the unit.

The program will remain active once the status display is closed. The symbol in the task bar displays the current status.

- To display the status, double click the symbol shown in the task bar.

2.2 Calling up status/messages

When there is a change in the status of the Power Tower the symbol also changes in the task bar and a message appears.

- | | | | |
|---|-------------------------------------|---|-------------|
|  | Failure |  | Information |
|  | Notice |  | Notes |
|  | Trouble free operation (no message) | | |

- The message can be resolved by following the instructions given by the Unit Monitor.

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DÜRR DENTAL AG