LUNOS®

DE Lunos® Fluoridlack

EN Lunos® Fluoride Varnish

FR Lunos® Vernis fluoré

IT Lunos® Lacca al fluoruro

ES Lunos® Laca de fluoruro

CE 0124

MD

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Lunos® Fluoridlack

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EN Operating Instructions

Lunos® Fluoride Varnish

with 22600 ppm fluoride / 1 ml contains 50 mg sodium fluoride (REF: CPZ030A0150)

Varnish containing fluoride for tooth desensitisation for the treatment of enamel and dentine surfaces. Fluoride Varnish adheres to moist surfaces and tolerates residues of moisture and saliva. It is based on an ethanolic suspension of colophony with artificial aromas and xylitol.

Warning

Fluoride Varnish is a medical device. This product is only intended for professional dental applications and must only be used by trained specialists!

Indications

1. the treatment of hypersensitive teeth;
2. the treatment of sensitive root surfaces.

Warnings

Fluoride Varnish contains colophony. Do not use this product in cases of known hypersensitivity (allergies) to this ingredient of Fluoride Varnish. Colophony can impair the polymerisation and adhesion of composites. In line with its intended purpose, Fluoride Varnish has a sticky consistency. When the tube is used, material residue on the screw thread can cause the lid to stick. If necessary, wipe off any contamination with a cloth containing ethanol. Fluoride Varnish should not be used on patients suffering from ulcerative gums or inflammation of the oral mucosa. Fluoride Varnish is not intended for systemic treatment. In rare cases oedematous swelling has been reported, particularly after application to larger areas. On comparable products there have been very rare reports of shortness of breath in children suffering from asthma. Nausea may occur in patients with a sensitive digestive system after extensive treatment. In cases of hypersensitivity: remove the varnish with a soft toothbrush and dental floss, then rinse with warm water.

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Precautionary measures

Do not carry out any other treatments with preparations containing fluoride on the day of treatment. Do not take fluoride tablets for a few days after the treatment with Fluoride Varnish.

Do not swallow the varnish.

Dosage

Treatment of patients with milk teeth: approx. 0.25 ml Fluoride Varnish. Treatment of patients with mixed dentition or permanent teeth: approx. 0.40 ml Fluoride Varnish. Apply a thin film of Fluoride Varnish to the tooth structure. A thick film will detach easily and offers no therapeutical advantages. Empirically the requirements for a complete set of teeth are between 0.25 ml and 0.40 ml.

Application

Thoroughly clean any tartar from the treatment area. For improved results completely remove any excess moisture/saliva from the surface under treatment. Apply Fluoride Varnish evenly so that the entire surface is coated with a thin film. Moisten the application area (either through light rinsing or natural saliva flow) to ensure bonding of the Fluoride Varnish. Patients should be advised to avoid eating solid food, drinking alcohol, cleaning their teeth or using dental floss for a period of 4 hours after treatment.

Storage and shelf life

Storage at 4°C - 23°C. Do not use the product after its expiry date.

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DÜRR DENTAL ITALIA S.r.l.