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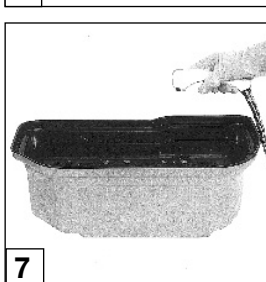
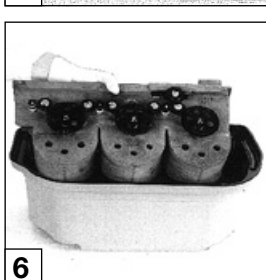
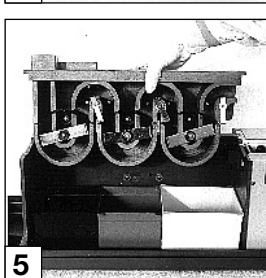
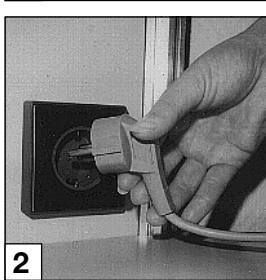
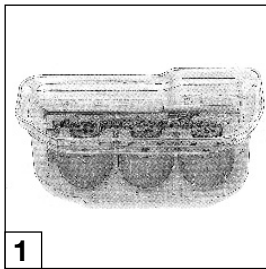
2. Cleaning

Every 6 months clean film transporter!

Before starting pull out mains plug!
Ensure the room is well ventilated and do not inhale the fumes!
Wear safety glasses and protective gloves!

- Position cleaning bath on a level and stable surface as close as possible to a wash basin.
- Carefully pour the contents of 2 bottles **-Component 1-** into cleaning bath. Do not splash!
Splashes on the skin or eyes must be immediately rinsed with water.
- Both pouches of **Component 2** powder must be **evenly** distributed over the whole length of the bath and then with a clean object, e.g. stirring rod, stir until all the powder has dissolved.
- Remove the Periomat cover.
- Take out film transporter and rinse thoroughly under running water.
- Carefully lay the transporter in the bath with the cogs facing upwards.
- Finally top up with water to the lower level mark and gently give the solution a short stir.
- The cleaning action takes approx. 20 mins. During this time you should periodically rotate the cogs.
- In the meantime change the Developer and Fixer solutions, see Directions for use Periomat "Preparing chemicals".
- After 20 mins. drain the chemicals from bath and remove film transporter, rinse under running warm water and clean with brush.

Please note, residue on the film transporter can destroy the fresh chemicals in the Periomat.

- Universal Cleaner complies with current drain water regulations.
- Place the film transporter on drying towel and leave to remove final drops. Then re-insert film transporter in Periomat.
- Clean the empty bottles which can then be recycled.

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